



Full Face-Mask/Cushions and Nasal-Mask/pillows/Cushions

- Replace accordingly to insurance allowance
- Wash daily: warm soapy washcloth or unscented baby wipe. Cleaning after usage by removing excess facial oil/dirt/make-up helps prolong the life of your mask.
- No harsh cleaning chemicals (disinfecting wipes)

Tubing:

- Replace accordingly to insurance allowance
- Wash weekly with warm soapy water and rinse thoroughly, hang to air dry.

Filters:

- Replace accordingly to insurance allowance
- Clean/dust off once a month.

Humidifier:

- Replace accordingly to insurance allowance
- Fill to max fill line (do not overfill) daily or as needed with distilled water.
- Wash weekly with warm soapy water and rinse thoroughly, leave open to air dry.

Headgear/Chin Strap:

- Replace accordingly to insurance allowance
- Wash weekly or as needed with warm soapy water and rinse thoroughly; let air dry thoroughly.

CPAP Machine:

- Keep back of machine clear from dust and debris.
- Keep machine and power cord free from blockage.